


# COREX™


BY TRINITY ORTHOPEDICS

- » Intuitive single-use device for percutaneous harvest of autologous bone graft (Gold Standard).
- » Proprietary capture mechanism rapidly harvests, stores, and delivers compressed cancellous bone dowels.
- » Allows for significant reductions in procedural time, cost, and risk of infection.
- » 9mm & 7mm sizes for minimally invasive harvesting from:
  - ◇ iliac crest
  - ◇ proximal or distal tibia
  - ◇ calcaneus
- » Reimbursable procedure codes:
  - ◇ CPT-20900
  - ◇ CPT-20902
  - ◇ CPT-20937 (spine only)




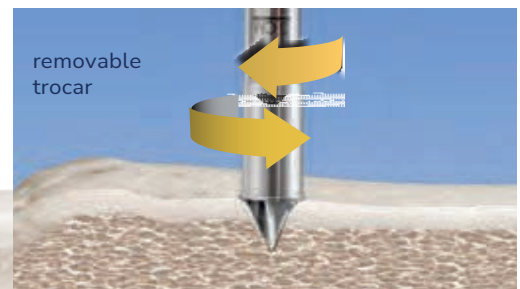

COREX™ features a non-aggressive distal tip for micro-fracturing cancellous bone and mitigating risk of cortical bone penetration.

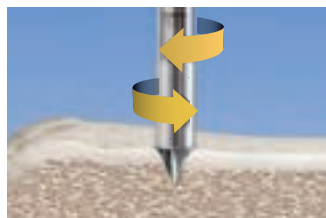


**Removable Trocar**  
Creates small cortical access window.

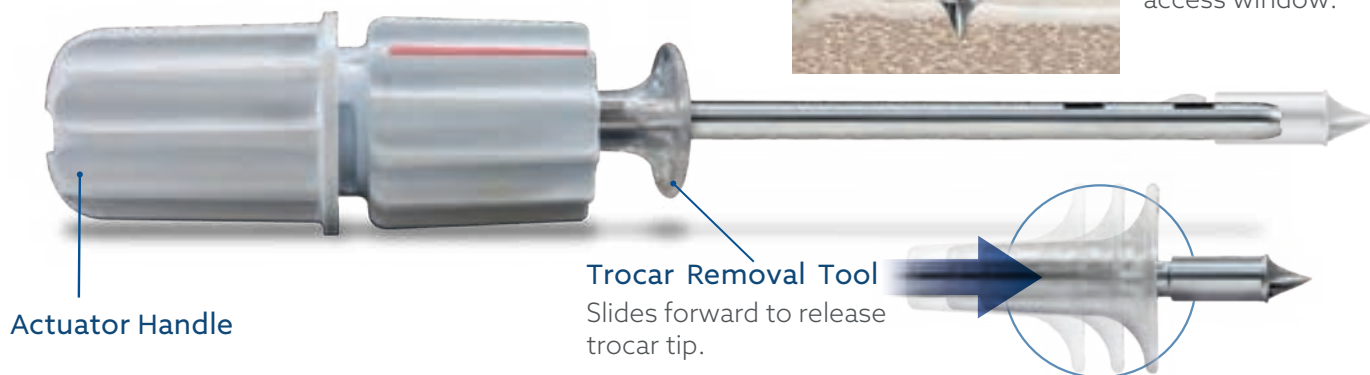

**OPEN POSITION: HARVESTS + COLLECTS** cancellous bone dowels within barrel of trephine.



Rotate upper handle **counterclockwise** until **GREEN + BLACK** indicator lines are aligned.




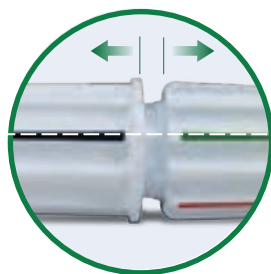


Exposes non-aggressive tip and **DEACTIVATES** bone graft capture.

**CLOSED POSITION: CAPTURES + RETAINS** cancellous bone dowels during withdrawal from harvest site.



Rotate upper handle **clockwise** until the **RED + BLACK** indicator lines are aligned.

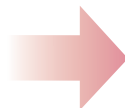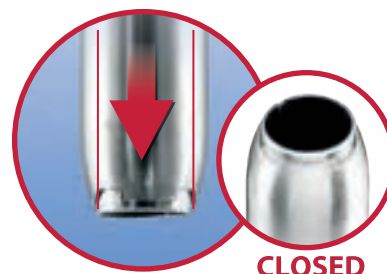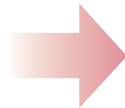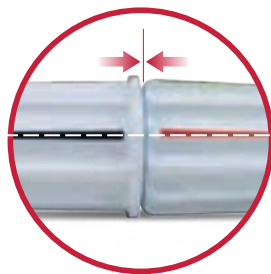

Exposes inner retaining sleeve and **ACTIVATES** bone graft capture.